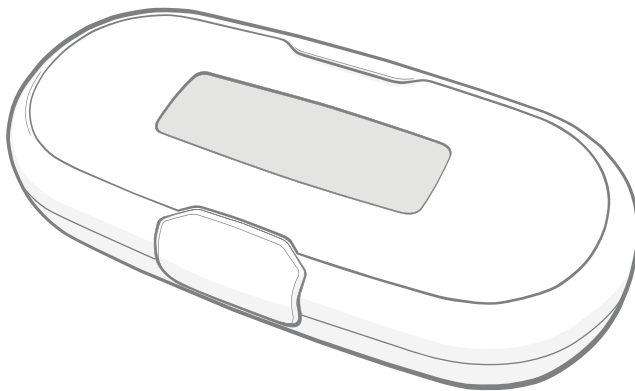
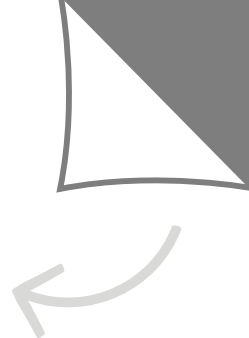
# Universal Microchip Reader

Model no. HRPG1N

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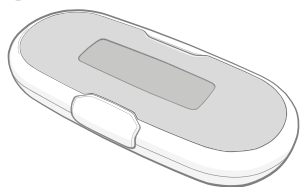
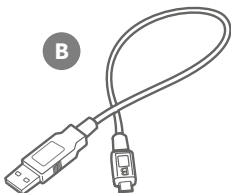
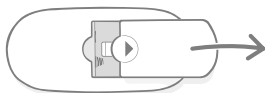
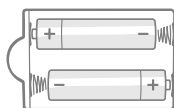
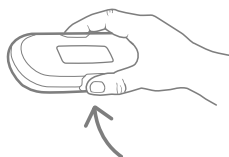
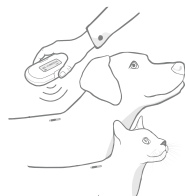
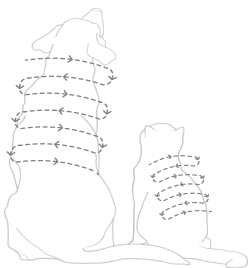
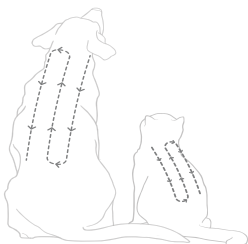
B

C

# An introduction to your new Universal Microchip Reader

## 1 What's in the box?

Please make sure you have all these parts:

- A** Universal Microchip Reader.
- B** USB cable (for firmware updates only).

## 2 Installing the batteries

- A** Remove the battery cover by pressing the arrow and sliding it away from the centre.
- B** Insert the batteries making sure the - and + symbols are the correct way round.

## 3 Scanning a pet

- A** Press the button on the reader to turn it on. (It will automatically turn off when not in use).
- B** Press the button again and move the reader around the animal, following one of the motion patterns detailed below to locate the microchip.
- C** When the reader detects a microchip it will display its number and, if applicable, microchip temperature.

## 4 Motion patterns

- A** **Scanning Pattern 1** - Side to side slowly across the animal's body.
- B** **Scanning Pattern 2** - Up and down slowly across the animal's body.

## 5 Display

- A** Microchip 15-digit number.
- B** Battery level indicator.
- C** Microchip Temperature (Only for temperature-sensing microchips).

## Important information

**It is your responsibility to exercise discretion and observe all safety measures required by law and common sense. Please read the following information carefully and retain this manual for future reference.**

- Your Universal Microchip Reader is for indoor use only.
- The Universal Microchip Reader requires 2 good quality 1.5 V LR6 alkaline “AA” cells to operate.
- The product is not designed for use with rechargeable batteries.
- Always change both batteries at the same time and avoid mixing battery types or manufacturers.
- Non-rechargeable batteries are not to be recharged.
- To avoid the risk of fire always take care to ensure batteries are fitted the correct way round as marked on the Universal Microchip Reader and never short circuit the battery terminals.
- There is a risk of explosion if the battery is replaced by an incorrect type. Always use good quality batteries from a reputable battery manufacturer to ensure the best performance.
- Exhausted or out-of-date batteries must be removed and disposed of properly in accordance with local legislation.
- If the appliance is to be stored unused for a long period, the batteries must be removed.
- There are no user serviceable parts other than the battery in the enclosure.
- Clean unit with pet-friendly cleaning products and a damp cloth only, do not immerse in water.

## Compatible microchips

This is a universal microchip reader which can read the following microchip types:

- FDX-A Non-ISO microchips
- FDX-B 15 digit microchips
- Trovan microchips
- Avoid microchips
- Thermochip

This reader fully reads FDX microchips which are compliant to the ISO11784/11785 standards.

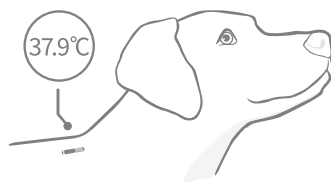
### What are temperature-sensing microchips?

Temperature-sensing microchips are primarily identification microchips, compatible with ISO 11784/11785 and universal readers. However, they also incorporate a temperature biosensor, capable of measuring an animal's temperature at the implantation site.

Temperature-sensing microchips are a non-invasive way of frequently measuring a pet's temperature, without causing stress. Multiple measurements taken consecutively can establish a trend of individual animal profile or average temperature and variability. Atypical variations from this trend can be indicative of changes in an animal's well-being.

**Microchip temperature is not a replacement for rectal temperature**, and should not be interpreted as such. Microchip Temperature and rectal temperature are not equivalent and should not be expected to show identical readings.

If an animal with a temperature-sensing microchip is scanned with the Universal Microchip Reader, its internal Microchip Temperature will be visible on the LCD screen alongside their microchip number.


## LCD display

The standard display for the reader is:

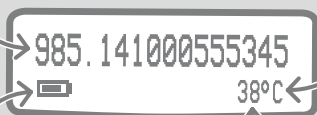**The animal's microchip number**

**Battery level indicator**

**Microchip Temperature**

(only for temperature-sensing microchips)

One of the icons below will replace the temperature reading if it is **out of range**:

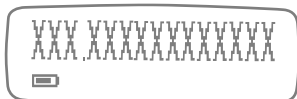
Below  
33°C (91.4°F)


Above  
43°C (109°F)

Displayed when the reader is **searching** for the pet's microchip.

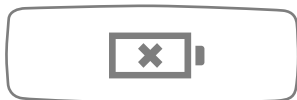
The scan will time out automatically if no chip is found and the chip number will display as a series of X's. Pushing the button again will activate another scan.

Displayed when the **batteries are completely drained**. To insert new batteries see **page 1**.

## Changing the temperature unit between °C and °F

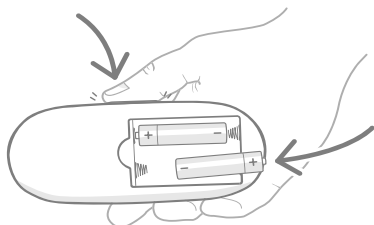To change the temperature setting of the reader, first remove the batteries. Then press and hold the reader's button while re-inserting the batteries. The temperature will now be displayed in Fahrenheit. To revert to Celsius, repeat the process.

## Warranty and disclaimer

**Warranty:** The Universal Microchip Reader carries a 2-year warranty from the date of purchase, subject to proof of purchase date. The warranty is restricted to any fault caused by defective materials, components or manufacture. This warranty does not apply to products whose defect has been caused by normal wear and tear, misuse, neglect or intentional damage.

In the event of a part failure due to faulty parts or workmanship, the part will be replaced free of charge during the warranty period only. At the manufacturer's discretion a replacement product may be provided free of charge in the case of a more serious malfunction. Your statutory rights are not affected.

If you are a resident of a jurisdiction with consumer protection laws and regulations that contain provisions that are more beneficial to you than the terms of this warranty, such laws and regulations shall supersede the terms of this warranty.

**Disclaimer:** The Universal Microchip Reader, including its use to measure microchip temperature is not intended to replace traditional veterinary advice. If you think you need veterinary advice you should take your pet to a veterinary practice. We expressly disclaim any liability for any loss, damage or costs that you may incur as a result of using the product as a substitute for professional veterinary advice. Furthermore we do not accept any liability for any veterinary fees that you may incur as a result of your use of the product.

Body temperature is not enough to determine disease alone. Microchip temperature is not a replacement for a rectal temperature measurement by a veterinarian, who is the only person qualified to determine your pet's health condition. Microchip temperature and rectal temperature, while correlated, are not equivalent and should not be expected to show identical readings.

**Disposal of old Products and used Batteries:** Either of these symbols on a product indicates it is subject to the following legislation:

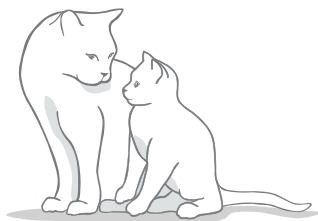
The Waste Electrical and Electronic Equipment Directive 2012/19/EU (WEEE Directive) and UK regulation SI 2013/3113 (WEEE 2013) and The Battery Directive 2006/66/EC (as amended 2013/56/EU) and the UK regulation SI 2009/890 requires households to dispose of EEE products and waste batteries that have reached the end of their life in an environmentally responsible manner. To reduce the impact on the environment these cannot be placed in household waste bins and must be recycled at an appropriate recycling facility.

Ensure batteries are replaced before the expiry date shown on the packaging or battery itself, and disposed of safely and appropriately.

In case of battery leaks, wear appropriate protective clothing to protect yourself from any exposure to harmful chemicals before disposing of the batteries and cleaning any spillage. When cleaning a spillage, avoid contact with skin. Battery chemicals shouldn't be placed near the eyes or ingested. Contact a medical professional immediately if this should occur.

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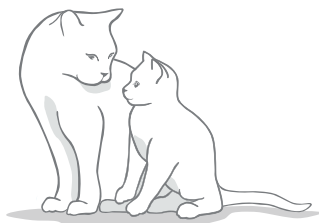
[surepetcare.com/support](https://surepetcare.com/support)

Copyright ©2023 Merck & Co., Inc., Rahway, NJ, USA, and its affiliates. All rights reserved. Ground Floor Building 2030, Cambourne Business Park, Cambourne, Cambridge CB23 6DW, United Kingdom.

HRPG1N

Jan 2023

01837-EU\_03

EU Authorised Representative: Allflex Europe, 35 Rue des Eaux, 35500 Vitré, France

Hereby, SureFlap Ltd. trading as Sure Petcare, declares that the radio equipment type HRPG1N is in compliance with the EU Directive 2014/53/EU and UK Radio Equipment Regulations 2017 (as amended).

The full texts of the declarations of conformity are available at the following internet address [www.surepetcare.com/ecdoc](https://www.surepetcare.com/ecdoc).

RF Identification: 126 & 133 kHz ; Maximum Radio Frequency Power: 47.9 dB(μA/m) @3m

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